



sune
KOS

Body

The precise combination of
Hyaluronic Acid and Amino Acids
**promotes the regeneration of the
Extra Cellular Matrix counteracting the
body's stretch marks and skin laxity of:**

Arms Inner Thigh Knees Abdomen

PLAAT
FORM

HY6AA

NO LOOKALIKES



Body

THE PRODUCT


Sunekos Body is a 10 ml solution containing low density Hyaluronic Acid (HA) + Amino Acids (AA) (HY6AA Formula: Glycine, L-Proline, L-Lysine, L-Leucine, L-Valine, L-Alanine) specifically balanced for the dermis of the body.^[1]



Sunekos Body helps with the production of collagen type IV and elastin, promoting the assembly of the Extra Cellular Matrix (ECM) components.


The dermis is different dependant on the region of the body.

This means that the

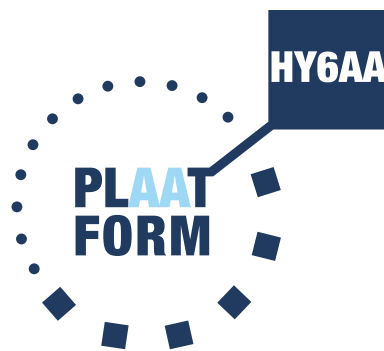

has been adapted considering the different density of the area.


Sunekos Body is an injectable indicated for the treatment of **stretch marks** and **skin laxity** of

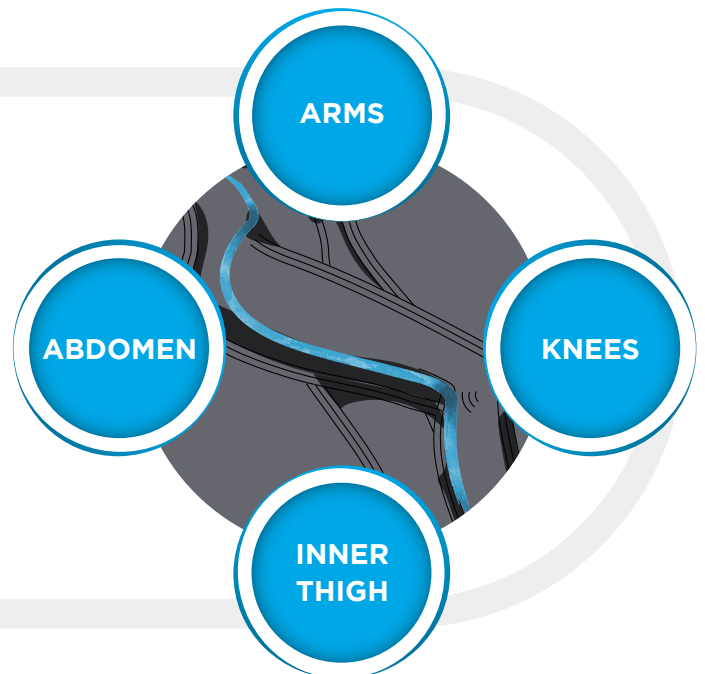




STRETCH MARKS HISTOLOGY

Body

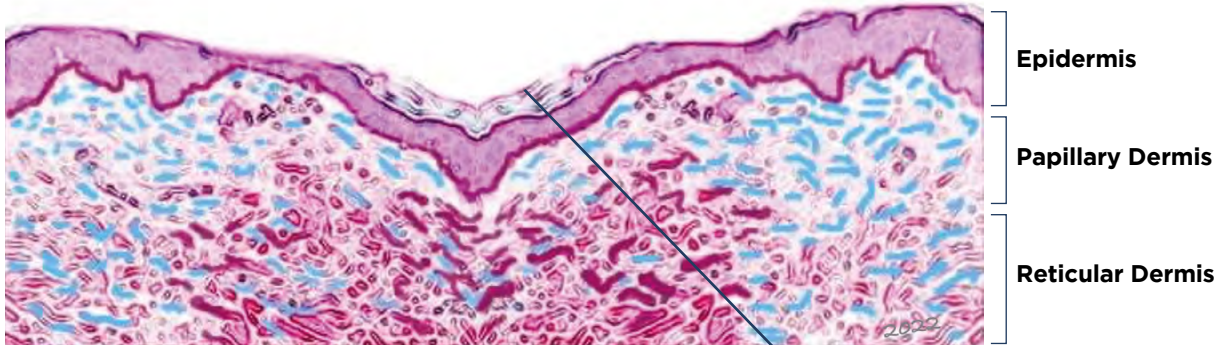

Drawing of skin cut with SD Alba - Elena Fasola 2022

— Elastin — Collagen

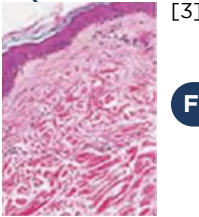


Stretch mark "Alba"




[2a] [2b]

Thigh stretch marks ^[3]

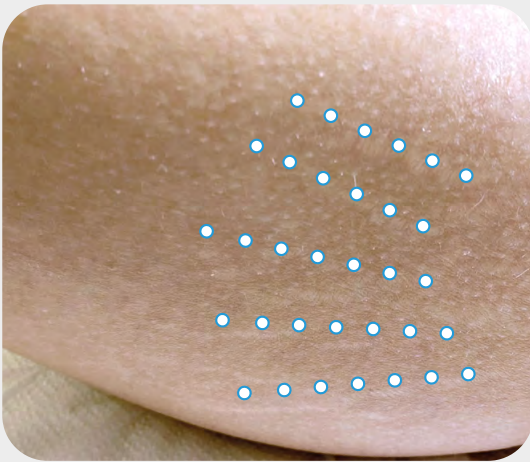

- Needle 30-32G and 12-13mm
- 30°-40° micro-bolus technique [fig.1]
- Series of 3 bolus at 3 different depth: superficial, medium and deep dermis [fig.2]
- 0,2 ml per bolus
- Injection is performed directly into the dermis, in the wall of the stretch mark
- N. of injections depends on the stretch mark dimension
- X 3 or 4 times, once every 15 days

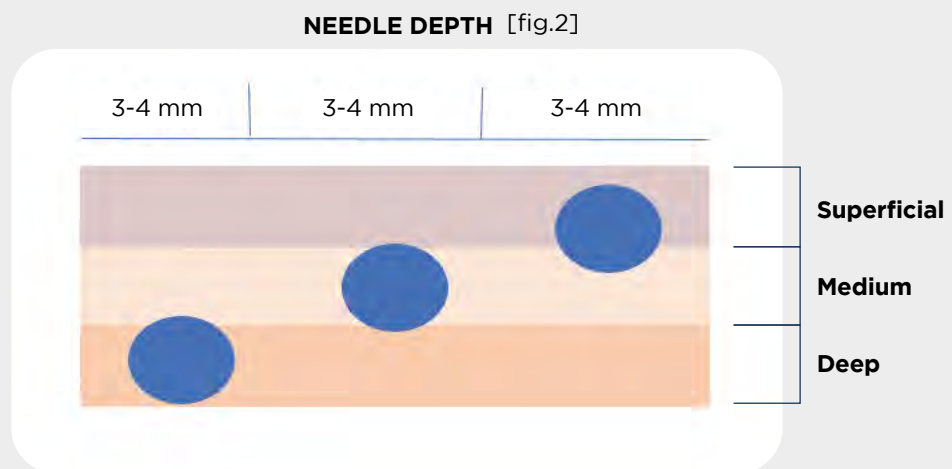

Standard Protocol



SKIN LAXITY

Skin laxity is a skin sign whose cause doesn't reside only in the skin, even if the dermis is always compromised.




After a precise and in-depth study of the patient, cutaneous laxity can be countered with combined therapeutic programs and / or procedures but Bio-stimulation first:

- **Biostimulation**
- Diet and Integrative Medicine path
- Gymnastics
- Ablative and non-Ablative Radiofrequency
- Ultrasounds
- Sometimes Filler

➔ **BIO-RESTRUCTURING FIRST!**

CLINICAL EVALUATION

21 patients injected 4 times, once every week using only 5 ml on arms and 5 ml on legs. This means that the trial has been performed using only 1/2 vial per area. [1]

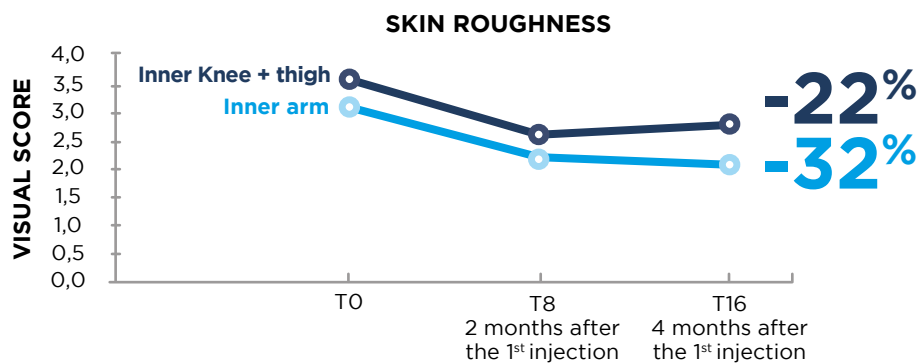

SKIN LAXITY

Improvement corresponding to a reduction of the visual score of at least 1 grade




Improvement measured on photographic scale



SKIN DENSITY

(profilometry)


* Holm-Sidak Adjusted t test p<0.05 vs T0

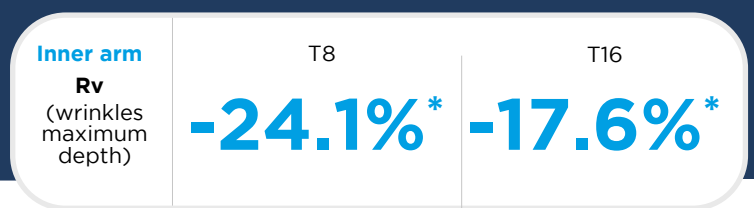

Measured with Primos compact portable (GFMesstechnik) equipment.

ACROSS TENSION LINES TECHNIQUE

Arms 2.5 or 5 ml x arm

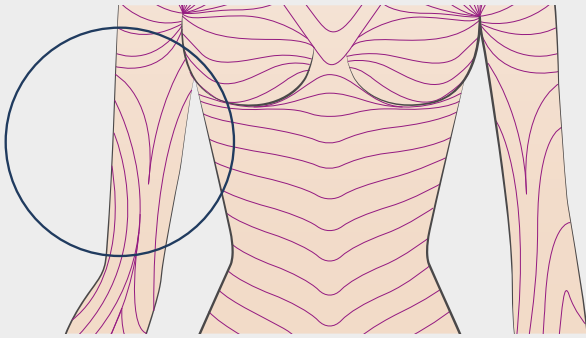


**sune
KOS**

15 - 21 days


3x

Body

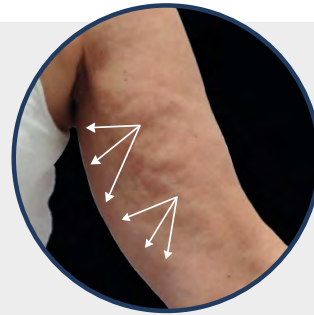

CANNULA


0.3 ml/point
1 or 2 areas x arm
(2.5 ml each)

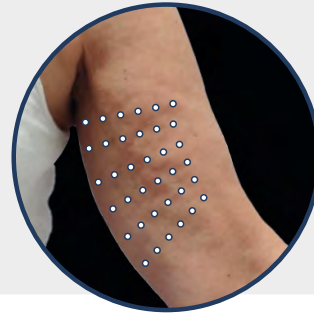

NEEDLE

0.1 ml/point
4-6 points x line

**Abdomen 5 ml upper abdominal part
5 ml lower abdominal part**

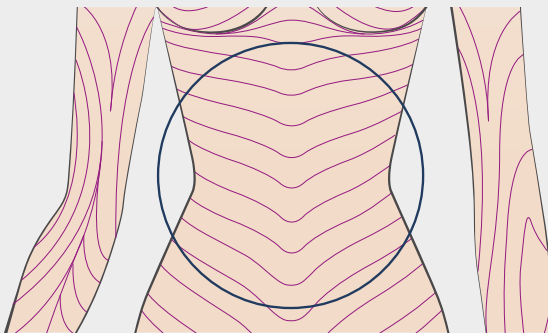


**sune
KOS**

15 - 21 days


3x

Body

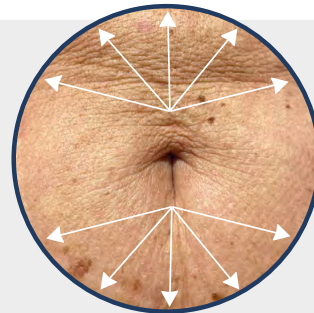

CANNULA


1 ml/point
10 ml tot

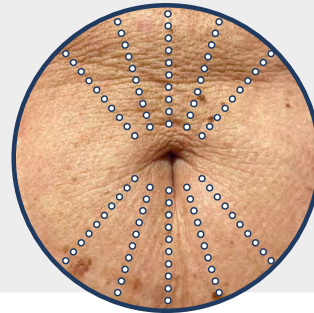

NEEDLE

0.1 ml/point
10 points x line
(10 ml tot)

Legs 5 ml x part Thigh & Gluteus

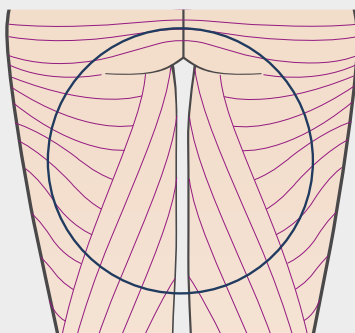


**sune
KOS**

15 - 21 days


3x

Body

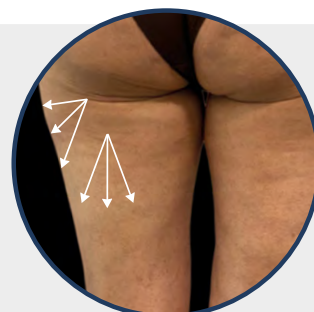

CANNULA

1 ml/point
10 ml tot x side

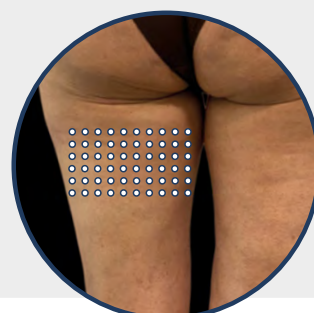

NEEDLE

0.1 ml/point
6-10 points x 10 lines
(max 10 ml x side)

THIGH STRETCH MARKS

BEFORE



AFTER



60 years old

ARMS SEVERE LAXITY

BEFORE



AFTER



61 years old

THIGH / LEG

BEFORE



AFTER



60 years old

ABDOMEN

BEFORE



AFTER



56 years old

Courtesy of Dr. Elena Fasola

Sunekos Body



Registered as
CE Medical Device Class III



One pack of Sunekos Body
contains 1 treatment (10 ml)
to be reconstituted

- 1x vial Sodium Hyaluronate
- 1x bottle Amino Acids

Bibliography

- [1] Sparavigna A. Skin Laxity a new intradermal injective treatment - Prime Journal March/April 2020
- [2a] Wang F. et al. Marked disruption and aberrant regulation of elastic fibres in early striae gravidarum - Br J Dermatol. Dec. 2015
- [2b] F.MacGregor J.L., MD Naissan, MD Wesley O. Striae distensae (stretch marks) - Uptodate 2021
- [3] Fasola E. Hyaluronic acid added to amino acids in the treatment of atrophic striae distensae (striae alba) - On going study 2023

esthetic med
arzt- und spitalbedarf

esthetic med GmbH
Chli Ebnet 3 - 6403 Küssnacht am Rigi - Switzerland
Tel: +41 44 802 33 88 - Email: info@estheticmed.ch
www.estheticmed.ch



Via Ciro Menotti, 1/A - 20129 - Milan - Italy
Tel: +39 02 744020
www.professionaldietetics.com
Email: info@pd-international.com

