COLOUR CHART
BIOCHROMADERM®

2023

BIOTIC PHOCEA
Laboratoires

BIOCHROMADERM[®],
the only medical pigment
in the world since 1999.

BREAST AREOLAS

4 – 9

SCARS

10 – 11

HAIRY AREAS

12 – 13

SUPPLEMENTS

14 – 15

SUMMARY OF COLOURS

16 – 17

DATA SHEET

18 – 19

BIOCHROMADERM®

Medical pigment
 Medical device CEIIb
 DMI - 2 ml.

The choice of formulation of our pigments guarantees results that will fade over time in order to respect the natural ageing of the skin.

Hypoallergenic, without preservatives, water or petrochemical derivatives, BioChromaDerm® pigments are sterilised by gamma ionisation at a dose higher than 25 kGy.

The only pigment authorised for tattooing for medical purposes, BioChromaDerm® is CEIIb certified, which is unique in the world.

Sold in a peelable bag in an individual box with implant card, information leaflet and 3 tracking labels (batch number,

This ready-to-use kit is packaged to perfectly fit the regulations imposed on hospitals and clinics for DMI.

Available in a wide range of colours, BioChromaDerm® allows a perfect adaptation of the colour of the pigmented area for a natural result.

BioChromaDerm® is available in several colours to ensure that you get the best possible result whatever the patient's phototype and the area treated. You will be able to perform chromatic reconstructions of breast areolas, camouflage of vitiligo, scars, stretch marks, burns and correct all other skin achromias. Hairy areas such as the skull, eyebrows, beard and genitals can also benefit from these devices to recreate density, hide a scar or complete a hair transplant.

The purpose of this document is to present all the shades in our range by type of use and to help you make your choice according to the type of skin of your patients and the expected result.

However, it is necessary to note that the result of a pigmentation does not only depend on the pigment you are going to use, but also on the technique you are going to use, the depth of the implantation, the lifestyle of your patient (sun exposure, smoking/ alcohol consumption, swimming, treatment, hormones...), but also on the preparation of the skin and the maintenance of the pigmentation.

Finally, although exhaustive, this document cannot replace an in-depth training in colorimetry and reparative dermopigmentation, which will allow you to better understand the choice of colour and techniques.

For any request: formation@biotic.fr

· cold ·
PIGMENT

· warm ·
PIGMENT

Areolas

Specialized in reparative and medical dermopigmentation, BIOTIC Phoea Laboratories offer a wide range of colours for the pigmentation of breast areolas and scars, to enable you to meet all your patients' requirements.

Pinkish AREOLAS

LP52 - Amber Rose

Very natural mucous pink.
Also very beautiful in a mixture to brighten up all roses.

LP29 - Santal

Cold rosewood for pinkish-brown areolas.

TIPS

Lighten with AR10 if too dark.

LP11 - Old Rose

Very natural, deep pale pink to be mix with AR10 to work the areola and pure for the nipple if it is dark.

LP04 - Cherry Rose

Red for a strong result.
Recommended for nipples with a tendency to redness.

LP01 - Raspberry Rose

Warm red.
Recommended for pigmenting dark nipples.

Beige AREOLAS

AR08 - Venus Complexion

Very light beige for white skin.
To be used in combination with AR10 or WH01 for a better ageing of the colour over time.

AR18 - Peach Complexion

Light peachy beige with a warm tendency.
To be used on white and pinkish skin always mixed with AR10.

AR10 - Pink Ocher

Cool pinkish beige for light skin.
Use as a base for light and pink areolas on skin types that will have a tendency to warm up a colour, to avoid oxidation of the pigment over time.

AR22 - Golden Brown

Very warm dark golden beige.

AR10 | AR18

Brown AREOLAS

AR14 - Red Brick
AR14 - Red Brick

Warm brick brown, fairly deep. Ideal for Mediterranean skins.

TIPS

Add 2 drops of AR10 for a good ageing of the colour over time.

AR16 - Brown Velvet

Light brown.
For dark and coppery skin.

AR21 - Hazel Skin

Light beige brown.
To be used as a base on browned areolas on Mediterranean skin types.

AR11 - Caribbean Skin

Warm brown. To be used on dark or mixed skins which will tend to cool down the implanted colour.

TIPS

When mixed with AR10, it is possible to have a lighter to darker beige depending on the proportions of the mixture.

AR23 - Chocolate Brown

Darker brown than AR11, for use on black and mixed race skins.

TIPS

For the areolas of very dark skin, mix with 2 to 3 drops of EB32.

EB08 - Dark Brown

Dark brown for areolas of black and mixed race skin, when the areola is very dark.

EB32 - Darkest Cool Brown

Dark ash brown to be used to darken the AR23 to obtain a dark areola, mixed race type, black skins.

Orange AREOLAS

AR12 - Rosewood Complexion

Light beige with a warm tendency.
For the fairest of skin types Mediterranean skin.

TIPS

To be used with the AR10 for a good ageing over time.

AR13 - Medium Venitian

Intermediate beige for dark skins.

TIPS

To be used with the AR10 for a good ageing over time.

AR15 - Venitian Brown

Orange beige, to be mixed with AR10 for red and light skins.

AR24 - Dark Taupe

Cool taupe brown, ideal for correcting pink or orange areolas.
Recommended for fair skins in combination with AR10.

AR25 - Beigy Rose

Very warm, light orange beige.
To be used with the AR10 to cool it down.

LPO3 - Tea Rose

Orange pink.

TIPS

Mix with AR10 to work the areola and pure for the nipple if it is dark

Scars

Burns

Stretch marks

Achromias

COLOURS FLESH

AR08 - Venus Complexion

Very light beige for white skins.
To be used in combination with AR10 or WH01 for a better ageing of the colour over time.

TIPS

Ideal for stretch marks: 4 drops of AR08 + 4 drops of white with 3 drops of diluent.

AR10 - Pink Ocher

Cool pink beige for fair skins.
Can be used for scars or mixed with WH01 to lighten a colour too dark for scars or stretch marks.

AR18 - Peach Complexion

Light peachy beige with a warm tendency.

To be used on white and pinkish skin always mixed with AR10.

AR12 - Rosewood Complexion

Light beige with a warm tendency.
For the fairest of Mediterranean skins.

TIPS

To be used with AR10 for a good aging over time.

AR11 - Caribbean Skin

Warm brown. To be used on dark or mixed skins which will tend to cool down the implanted colour.

TIPS

When mixed with AR10, it is possible to have a lighter to darker beige depending on the proportions of the mixture.

AR23 - Chocolate Brown

Darker brown than AR11, for black and mixed race skins.

TIPS

For the areolas of very dark skins, mix with 2 to 3 drops of EB32.

EB08 - Dark Brown

Dark brown for the areolas of dark black skins.

CC08 - Rosy Corrector

Skin tone corrector for dewy skin.
Add 3 to 5 drops of thinner to 10 drops of concealer.

Hairy areas

Eyebrows

Beard

Head

Genitals

HAIRY AREAS

EB32 - Darkest Cool Brown

Dark ashy brown to be used for darkening.
Ideal for almost black hairs.

EB08 - Dark Brown

Dark brown for dark skin.
Ideal for obtaining a dark brown
eyebrow or hair.

TA13 - Charcoal

Intense black for black hairs.

Good to know

Other shades are available in our aesthetic range «Airless Color®».
These pigments are not medical devices and should only be used for aesthetic and not medical purposes.

Ask for your Airless Color® colour chart.

Supplements

WH01 - **Snowy White**

White to lighten a colour

D101 - **Sterile diluent**

Designed to lighten a colour without altering its shade. It can be used in a mixture with all BIOTIC Phocea colours.

TE01 - **Allergy Test**

The universal allergy test for all BIOTIC Phocea pigments. Make a painless test point behind the ear: wait one month to check the good tolerance of the product. Do not risk pigmenting patients with an allergic background.

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